


# LYMPHATREX Essential

## STOCKINGS

### Anatomical Markers

- Ⓒ is located at the gluteal fold (for the tights)
- Ⓕ is equidistant between Ⓒ and Ⓔ
- Ⓗ corresponds to the middle of the patella
- Ⓓ is located on the head of the fibula (3 - 4 cm below the patella)
- Ⓒ corresponds to the widest point of the calf
- Ⓖ corresponds to the start of the calf muscles
- Ⓖ to be measured at the narrowest point of the ankle
- Ⓕ goes through the instep and the tuberosity of the calcaneus
- Ⓐ goes through the head of the metatarsal bones (at the widest point)

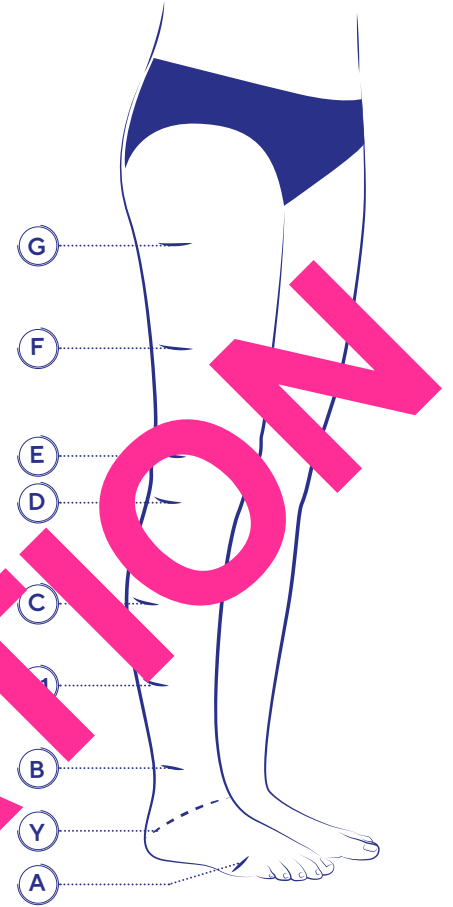

*All markers must be positioned on the external face of the leg.*

### Measurement Instructions

