THUASNE

Ligacast® Anatomic

fr	Attelle stabilisatrice de cheville	5
en	Stabilising ankle brace	6
de	Orthese zur Stabilisierung des Sprunggelenks	7
nl	Stabilisatiebrace voor de enkel	9
it	Tutore stabilizzatore per caviglia	10
es	Férula estabilizadora de tobillo	12
pt	Tala de estabilizadora de tornozelo	13
da	Stabiliserende ankelskinne	14
fi	Nilkkatuki	16
sv	Stabiliseringskena för ankeln.	17
el	Νάρθηκας σταθεροποίησης ποδοκνημικής	19
cs	Stabilizační ortéza kotníku	20
pl	Orteza do stabilizacji kostki	21
lv	Potītes stabilizēšanas saite	23
lt	Stabilizuojantis kulkšnies įtvaras	24
et	Hüppeliigese fikseerimise lahas	25
sl	Opornica za stabilizacijo gležnja	27
sk	Stabilizačná členková dlaha	28
hu	Stabilizáló rögzítő bokához	29
bg	Описание/Предназначение.....	31
ro	Atelă stabilizatoare pentru gleznă	32
ru	Стабилизирующий ортез для лодыжки	34
hr	Udlaga za stabilizaciju gležnja	35
zh	踝部稳定夹板	37
ar	جيرة تثبيت الكاحل	39

	0	< 160 cm
	1	> 160 cm

fr	Taille du patient
en	Patient height
de	Größe des Patienten
nl	Maat van de patiënt
it	Altezza del paziente
es	Tamaño del paciente
pt	Tamanho do paciente
da	Patientens højde
fi	Potilaan koko
sv	Patientens storlek
el	Μέγεθος ασθενούς
cs	Výška pacienta
pl	Rozmiar pacjenta
lv	Pacienta augums
lt	Paciento dydis
et	Patsiendi suurus
sl	Velikost pacienta
sk	Veľkosť pacienta
hu	A beteg mérete
bg	Размер на пациента
ro	Talia pacientului
ru	Размер пациента
hr	Veličina pacijenta
zh	患者身高
ar	مقاس المريض

fr	Stabilisation
en	Stabilisation
de	Stabilisierung
nl	Stabilisatie
it	Stabilizzazione
es	Estabilización
pt	Estabilização
da	Stabilisering
fi	Stabiloi
sv	Stabilisering
el	Σταθεροποίηση
cs	Stabilizace
pl	Stabilizacja
lv	Stabilizēšana
lt	Stabilizavimas
et	Stabiliseerimine
sl	Stabiliziranje
sk	Stabilizácia
hu	Stabilizálás
bg	Стабилизация
ro	Stabilizare
ru	Стабилизация
hr	Stabilizacija
zh	稳定
ar	التثبيت

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STABILISING ANKLE BRACE

Description/Destination

This device is a ankle brace providing joint stabilisation.
The device is intended only for the treatment of the indications listed and for patients whose measurements correspond to the sizing table.

Composition

Rigid components: polypropylene - thermoplastic polyurethan.

Textile components: thermoplastic elastomer - polyamide.

Properties/Mode of action

Rigid plastic lateral shells ①.

Honeycomb pads perfectly contour to ankle's anatomy ②.

Precise fit thanks to adjustable straps ③.

Pivoting heel straps maintain plantar and dorsiflexion ④.

Heel loop with width adjustment ⑤.

Indications

Moderate to severe ankle sprain.

Ankle immobilisation.

Conservative treatment of Achilles tendon injuries.

Contraindications

Do not use the product if the diagnosis has not been confirmed.

Do not use in the event of known allergy to any of the components.

Do not apply the product in direct contact with broken skin.

History of venous or lymphatic disorders.

Precautions

Verify the product's integrity before every use.

Do not use the device if it is damaged.

Choose the appropriate size to fit the patient, referring to the size chart.

It is recommended that a healthcare professional supervises the first application.

Strictly comply with your healthcare professional's prescription and recommendations for use.

Check the condition of the affected limb daily (with particular attention for patients with sensory deficit).

In the event of discomfort, significant hindrance, pain, variation in limb volume, abnormal sensations or change in colour of the extremities, remove the device and consult a healthcare professional.

For hygiene and performance reasons, do not re-use the product for another patient.

Do not use the device in case of application of certain products on the skin (creams, ointments, oils, gels, patches...).

Do not wear the product in a medical imaging machine.

Do not wear the product when driving a vehicle.

The systematic use of a long sock is recommended when wearing the device.

It is recommended to adequately tighten the device to achieve support/immobilisation without restricting blood circulation.

Undesirable side-effect

This device can cause skin reactions (redness, itching, burns, blisters...) or wounds of various degrees of severity.

Possible risk of venous thrombosis.

Any serious incidents occurring related to the device should be reported to the manufacturer and to the competent authority of the Member State in which the user and/or patient is resident.

Instructions for use/Application

From a sitting position pull on sock with the foot on the floor **A**.

Step into the ankle brace making sure the shells are properly centered along the ankle **A**.

Fasten the lower strap wrapping it around the brace **B**.

Adjust the strap and fasten securely to the self-fastening surface **B**.

Adjust the small posterior strap using the self-fastening tabs **B**.

Adjust the width of the heel strap if necessary **C**.

Fasten the upper strap wrapping it around the brace **D**.

Carefully slip the foot into a shoe **E**.

The ankle brace may be taken off to allow the foot to be washed.

Only put it back on once you have properly dried the foot again.

Care/Maintenance

Product can be washed in accordance with the instructions shown on this leaflet and on the label. Wipe with a damp cloth. Do not use detergents, fabric softeners or aggressive products (products containing chlorine). Do not dry clean. Do not tumble-dry. Do not iron. Squeeze out excess water. Dry flat. Dry away from any direct heat source (radiator, sun, etc.).

Storage

Store at room temperature, preferably in the original packaging.

Disposal

Dispose of in accordance with local regulations.

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Bei Bedarf den Gurt unter der Ferse in der Breite einstellen ©.
Thuasne SAS - SIREN/RCS Nanterre 542 091 186 - capital 1 950 000 euros